



Addendum

to

EnteraLite® Infinity® Operator's Manual

Part Number 25356-001

Revisions A thru G

Seriously health compromised users and children, or users who are extremely sensitive to receiving air with an enteral feeding, should not use the Infinite Dose or Interval Feeding programming option listed in the operator's manual. It is recommended that a single feeding regimen for specific dose be used for this particular user group.

DIRECTIONS FOR USE > Operating the Pump: A Single Feeding Example. Pages 10-12

The specific dose regimen is extremely important when a licensed healthcare clinician prescribes the use of feeding liquids other than commercially available enteral formulas or human milk, such as, but not limited to, blenderized whole foods. Use of this type of nutrition may cause sticky solids which could cause an obstruction of the optical sensor. It is recommended that all users clean and maintain the pump and optical sensors according to manufacturer's recommendations.